HYDRIM[®] M2

- **Bedienerhandbuch**
- **Operator's Manual**
- **Mode d'emploi**
- **Manual del operador**
- **Manuale d'uso**

SciCan

A HIGHER STANDARD

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Table of Contents

1. Introduction	32	6. Care and Maintenance	52
2. Important Information	33	6.1 Replacing the HIP Cleaning Solution ..	52
3. Disclaimers	34	6.2 Refilling the Water Softener	
4. Installation Instructions	35	Salt Reservoir	53
5. Instructions for Use	37	6.3 Changing the Air Filter.....	53
5.1 Instrument Reprocessing		6.4 Cleaning the Hydrim M2	54
Recommendations.....	37	6.5 Filter and Wash Arm Maintenance	54
5.2 Cycle Description Chart.....	38		
5.3 Baskets and Accessories	39	7. Troubleshooting.....	55
5.4 Activating the Hydrim M2	40	8. Material Safety Data Sheet	57
5.5 Interrupting / Aborting a Cycle	41	9. Warranty	59
5.6 Overview of the Hydrim M2 Menus	42	10. Specifications	60
5.7 User Menu	43	11. Serial Number Plate	61
5.8 Technician Menu	44		
5.9 Setup Menu	45		
5.10 Opening the Door in			
Case of Power Failure	50		
5.11 Printer / Data Logger Setup.....	51		

Manufactured by:
SciCan Ltd
1440 Don Mills Road
Toronto ON M3B 3P9
Canada
Local: 416.445.1600
Fax: 416.445.2727

For all service and repair inquiries:
EU: +49-(0)821-56-74-56-20
International: 1-416-445-1600
Canada: 1-800-870-7777
United States: 1-800-572-1211
Email: techservice.ca@scican.com

EU Representative:
SciCan GmbH
Kurzes Gelände 10
86156 Augsburg
Germany
Tel: +49-(0)821 56 74 56-0
Fax.: +49-(0)821 56 74 67-99

SciCan Medtech
Alpenstrasse 16
6300 Zug, Switzerland
Tel: +41(0) 41-727-7027
Fax: +41(0) 41-727-702

SciCan Inc.
701 Technology Drive
Canonsburg PA 15317
USA
Phone: (724) 820-1600
Fax: (724) 820-1479
Toll free: 1-800-572-1211

HYDRIM and STATIM are registered trademarks of SciCan Ltd. SystM and HIP are trademarks of SciCan Ltd.
All other trademarks referred to in this manual are the property of their respective owners.

1. Introduction

Thank you for selecting the Hydrim M2 instrument washer-disinfector. The Hydrim M2 is designed in accordance with ISO15883-1 and -2, the European standards for washer-disinfectors. The Hydrim complements the Statim autoclaves by quickly and hygienically preparing soiled instruments for sterilisation. To ensure years of safe, trouble-free service, carefully review this Operator's Manual before operating the unit. The Hydrim M2 is suitable only for the applications listed in this manual. Using the Hydrim for other purposes may be dangerous. The manufacturer cannot be held responsible for damage caused by improper use.

Operational, maintenance and replacement instructions must be followed for the product to perform as designed. All trademarks referred to in this manual are the property of their respective owners. Contents of this manual are subject to change without notice to reflect changes and improvements to the Hydrim product.

The Hydrim M2 should only be installed and serviced by a qualified contractor as it is an Installation Category 2 device. Before connecting the unit, the installer should check that the voltage and frequency of the electrical supply correspond with the details in the installation instructions. The unit must only be plugged into a grounded outlet. Please note that this unit is completely isolated from the electrical supply only when it is unplugged or the circuit breaker is turned OFF. This must be done before any repair work is carried out.

The Hydrim M2 uses HIP™ Cleaning Solution exclusively. The unit will not perform as described if other detergents / cleaning products are used. Other detergents / cleaning products may damage the unit, the instruments, cause excessive foaming, and void the warranty. Do not use solvents in this unit. Do not allow small children or unauthorised personnel access to this unit or its controls.

2. Important Information

Caution: Hot Surface and/or Hot Steam

Caution: Risk of electrical shock. Disconnect supply before servicing

The following symbols appear in this book.

Caution, there is a potential hazard to the operator.

This situation may lead to a mechanical failure.

This is important information.

The following symbols appear on the touchscreen:

Used to stop the cycle

Forward Arrow – used to move to the next menu

Back Arrow – used to move to the previous menu

Up Arrow – used to adjust a value up, or to move between screens in a menu

Down Arrow – used to adjust a value down, or to move between screens in a menu

Unlock – used to unlock the door

Printer – used to obtain a print out, if the Hydrim has a printer

Used to clear the error history

Info menu – allows access to User, Setup and Technician menus.

3. Disclaimers

Do not permit any person other than authorised personnel to service, maintain or supply parts for your Hydrim M2. SciCan shall not be liable for incidental, special or consequential damages caused by any maintenance or services performed on the Hydrim M2 by unauthorised personnel, or for the use of equipment or parts manufactured by a third party, including lost profits, any commercial loss, economic loss, or loss arising from personal injury.

Never remove the covers of the unit and never insert objects through holes or openings in the cabinetry. Doing so may damage the unit and/or pose a hazard to the operator. If the unit is used in a manner other than that specified, the protection provided by the equipment may be impaired. Avoid contact with eyes and mouth. Never lean on the open door. The unit may tip forward causing injury.

Always turn the unit **OFF** before adding softener salt, adding solutions or performing routine maintenance to the unit.

The owner shall not allow any personnel other than the trained and authorised personnel to operate the unit.

4. Installation Instructions

To open the wash chamber door, plug in the unit, turn it on and press the unlock symbol.

Tools and supplies required for installation:

- Slot screwdriver
- Adjustable wrench
- Pliers

Ensure that HIP Cleaning Solution (instrument wash chemical) is available. All other supplies are included with the Hydrim unit.

Specifications:

Voltage: 230 – 240 V	Height, built-in: 830mm
Frequency: 50 Hz	Height, free-standing: 850 mm
Rated load: 2 kW	Width: 600 mm
Circuit breaker: 13 A	Depth: 600 mm
Running Noise: 60 dBA	Depth with door open: 1200 mm
	Weight: 80 kg

Utility Hook-ups

Hose / Cord	Length / Diameter	Max. Distance from inlet / drain	Water Pressure	Shut-off valve
Hot inlet	1.9 m G 3/4"	1.5 m	2-5 bar	Yes
Cold Inlet	1.9 m G 3/4"	1.5 m	2-5 bar	Yes
Drain	1.5 m 3/4"	—	—	—
Electrical	1.8 m AWG 18-3	—	—	—

Installation Configurations:


If the Hydrim is installed in a Sterilisation Center, the manufacturer of the Sterilisation Center should allow a 10mm space at the top, back and both sides of the Hydrim. This will facilitate installation, levelling, and service access to the Hydrim.

Do not move the Hydrim into place by maneuvering the open wash chamber door. This may cause the door to become misaligned and leak.

Levelling the Hydrim:

The unit is standing on three supports: rollers (wheels) at the back and two legs at the front.

1. Remove the front kickplate. Push the Hydrim into place while lifting the strap at the front to allow the unit to roll on the rollers.
2. Adjust the front legs as required until the Hydrim is level.
3. The rear two legs are used only if the floor is uneven or cannot provide support to the rollers.
4. Tuck the strap under the machine before replacing the kickplate, so as not to interfere with the exhaust air vents.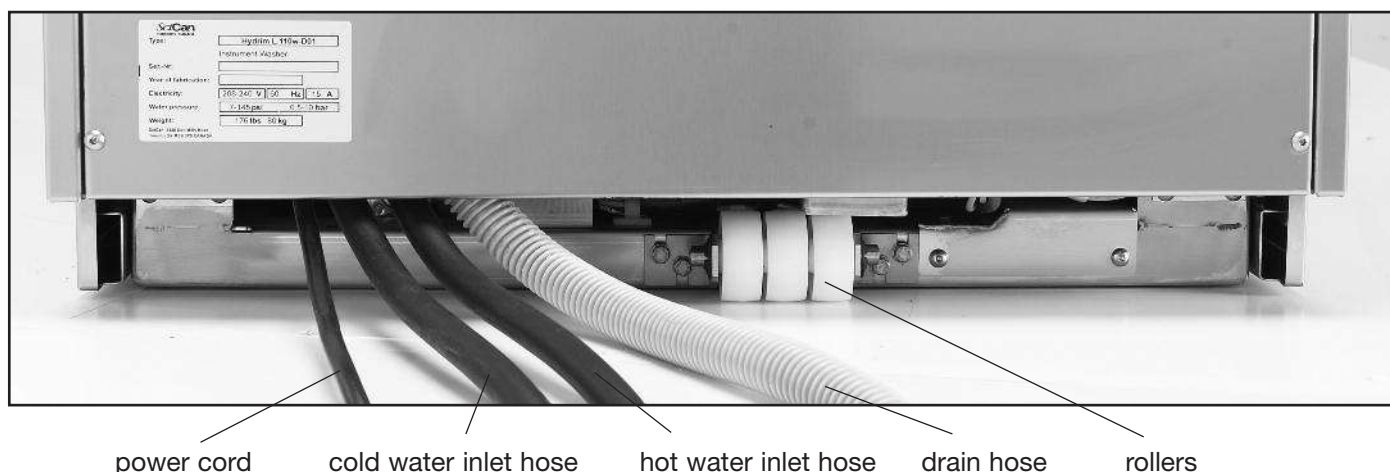

The electrical outlet should not be located directly behind the unit and should be accessible.

4. Installation Instructions

Connecting the water inlet hoses:

Connect the hot and cold water hoses to the hot and cold water supply. For optional installation with cold and deionised water supply, connect the cold water hose to the cold water supply and the hot water hose to the deionised water supply.

Drain requirements:

Connect the drain hose to the drain outlet. The drain hose can be attached to existing drain lines using a 3.5 cm or larger standpipe / P-trap combination. If the hose is connected directly to the drain line, fittings and adapters should not reduce water flow. The drain hose should be attached to the main drain at a point no more than 1 metre above the base of the Hydrim. A floor drain is acceptable (check local codes).

Electrical Requirements:


Connect the electrical cord to the power supply. The Hydrim must have a dedicated electrical circuit with a 16A fuse. The outlet must be grounded.

Additional Information:

- The Hydrim unit is heavy. Use a forklift or dolly and exercise caution when moving it.
- The Hydrim should be serviced on site.
- The Hydrim is equipped with an air gap / anti-suction device to prevent backflow of dirty water into the water supply. No other air gap device is necessary.
- If you need to extend the water inlet and drain hoses, ensure that you use commercial grade plumbing hose. The maximum length of the drain hose is 3.3 m / 13 ft.

Installing Cleaning Solution:

Install HIP Cleaning Solution as follows:

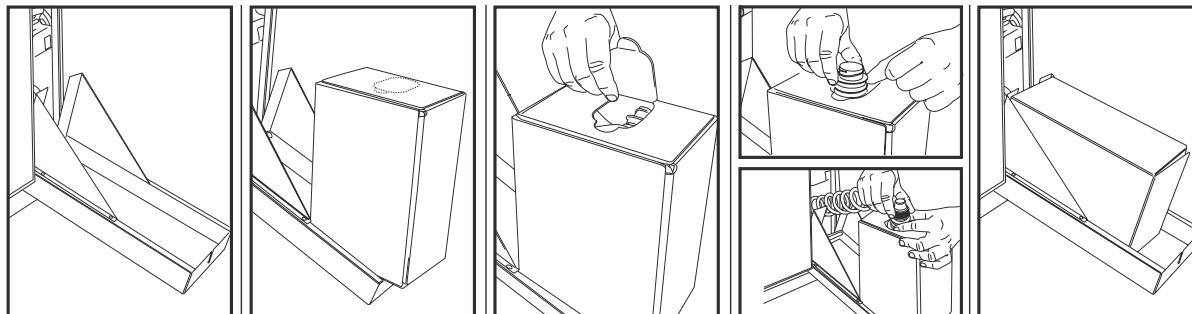Testing water hardness and setting the water softener (salt):

Hydrim is equipped with a built-in water softening system which needs to be adjusted according to the local water hardness. The Hydrim water test kit includes three water hardness test strips in bags. Take a water sample from the location where the machine will be installed. Open one of the bags and remove the test strip. Dip the strip in the water. Compare the colour of the strip with the chart on the back of the bag. Determine the water hardness according to the chart on the water test kit envelope. Power the unit on. Touch the “i” in the lower right hand corner of the screen. Select

“Setup” and then “Set Regeneration”. Using the up and down arrows, set the water softener regeneration level according to this table:

Hardness °dH	Hardness – ppm	Hydrim setting
0 - 6	0 - 110	0
6.7 - 7.8	120 - 140	1
8.4 - 10	150 - 180	2
10.6 - 11.8	190 - 210	3
12.3 - 16.2	220 - 290	4
16.8 - 20.7	300 - 370	5
21.3 - 30.3	380 - 540	6
30.8 - 49.9	550 - 890*	7
> 49.9	> 890	water treatment required

* consider using an additional water treatment

If the salt setting is 0, do not add salt. Otherwise, unscrew the salt container lid and pour 1 litre / 1 quart of water into the water softener. Add 1 kg / 2.2 lbs of water softening salt in the same manner. Screw the salt container cap on tightly.

Installation Test: Turn on the shut-off valves. Run a cycle, checking for leaks in the plumbing connections.

5. Instructions for Use

5.1 Instrument Reprocessing Recommendations

The Hydrim M2 is designed to clean and thermally disinfect (depending on the cycle chosen) general dental and medical instruments.

The disinfection level (P2 minimum $A_0 = 600$, P3 minimum $A_0 = 3000$) is intended to reduce the number of viable microorganisms on a product to a level appropriate for its intended further handling or use depending on requirements specified by regional or national authorities. Regulatory authorities may specify more stringent requirements within the territories for which they are responsible.

SciCan recommends terminal sterilisation after processing in the Hydrim M2.

When used correctly, the Hydrim M2 is effective for the removal of debris on instruments that may result in the insulation of microorganisms during terminal sterilisation. Prior to placing instruments in the Hydrim, consult the instrument manufacturer's reprocessing instructions.

Examples of medical and dental instruments suitable for cleaning and disinfecting in the Hydrim M2 include:

- Stainless steel scalers, spatulas and hemostats.
- Stainless steel retractors, needle holders, skin hooks, tissue forceps, scissors, curettes, and other stainless steel instruments free of deep cavities.

For best results, instruments processed in the Hydrim M2 should have the following properties:

- Heat resistance to a temperature of up to 96°C
- Corrosion resistance in the presence of heat and alkalinity.

! Microkeratomes, phaco tubing, fiberoptics and electrical equipment should not be processed in the Hydrim.

! HIP™ (Hydrim Cleaning Solution with Instrument Protection) has been carefully formulated to provide outstanding material compatibility with a wide range of metals and coatings. However, some manufacturers' instruments may not be compatible.

! When processing small or light objects in the Hydrim use the basket with lid (SciCan part number 01-109966S).

When processing dental handpieces in the Hydrim M2, follow the care and maintenance instructions of the handpiece manufacturer at the completion of the cycle.

5. Instructions for Use

5.2 Cycle Description Chart

	P1 – Regular Wash Cycle	P2 – Wash Disinfect 80°C	P3 – Wash Disinfect 90°C
Description	Use for instruments and cassettes when disinfection is not required	Use for instruments and cassettes when disinfection (minimum A ₀ of 600) is required	Use for instruments and cassettes when disinfection (minimum A ₀ of 3000) is required
Cold Prewash	<30°C	<30°C	<30°C
Wash	50°C 9 minutes	50°C 9 minutes	50°C 9 minutes
Rinse / Disinfect	60°C	80°C for 10 minutes	90°C for 5 minutes
Dry*	20 min. standard 30 min. intensive	15 min. standard 20 min. intensive	15 min. standard 20 min. intensive
Total Time** without drying	31 minutes	50 minutes	51 minutes
Water Consumption (without drying)	40 L	40 L	40 L

* Cassettes may be of a complex design that does not permit full dryness of the cassette inner surfaces

** Cycle times depend on the temperature of incoming water, mass of the load and mains voltage.

5. Instructions for Use


5.3 Baskets and Accessories

! Observe the instrument manufacturer's recommendations for cleaning and care. When loading the unit, arrange loose instruments so that they do not touch one another. Instruments with concave surfaces or cavities should be placed so that water can drain freely.


The following optional items are available for the Hydrim:


Rack for five 8" x 11" (203 x 280 mm) cassettes, for upper or lower trolley. Full size (1/1). Part-number 01-109963S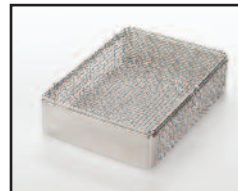

Basket with lid. Quarter size (1/4). Part-number 01-109966S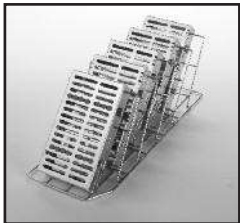

Rack for five medium size cassettes, for upper or lower trolley. Half-size (1/2). Part-number 01-109964S

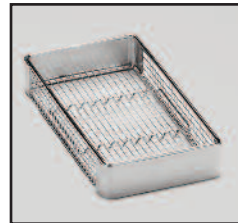Hygiene basket for loose instruments. Quarter size (1/4). Part-number 01-109967S

Rack for four 11" x 14" (280 x 368 mm) cassettes, for upper or lower trolley. Full size (1/1). Part-number 01-109965S


Statim 5000 basket. Part-number 01-107241


Hinged instrument rack. Part-number 01-110409S

Long Basket. Part-number 01-108232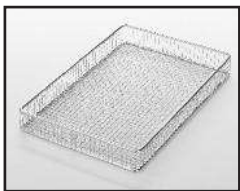

Statim 2000 basket. Part-number 01-107240

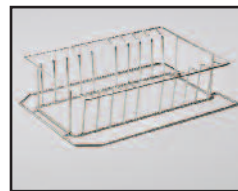Rack for half trays. Part-number 01-110412S

Rack for vertical instruments. Part-number 01-110411S


Handpiece / suction tube adapter rack. Part-number 17166.60

5. Instructions for Use

5.4 Activating the Hydrim M2

Prior to activating the washer-disinfector, check that the water supply is on and follow these steps: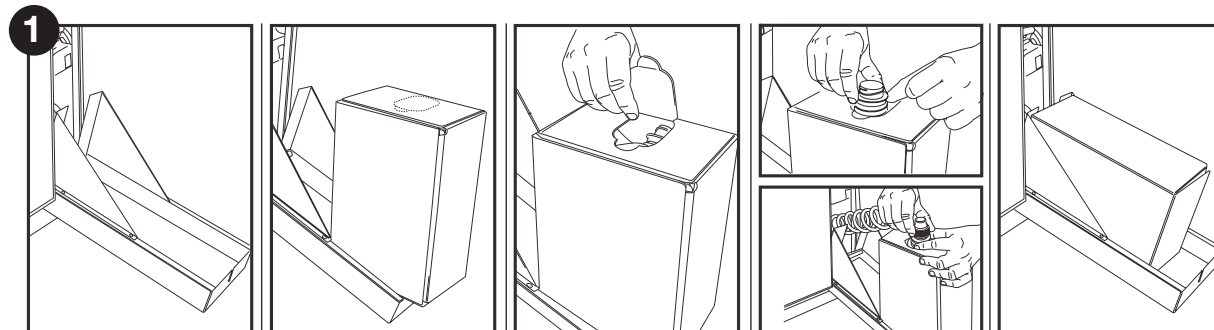

Open the chemical door and ensure that a bottle of HIP Hydrim Cleaning Solution has been installed.

Turn the unit on.


Touch the screen to begin.

Press to unlock door.


Load instruments into the baskets or cassettes, place them in the machine and close the door.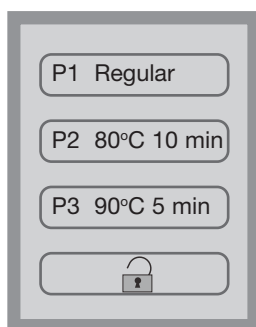

Select the desired cycle.


6

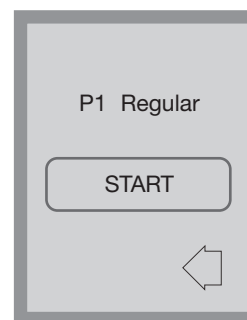
Press Start

7


5. Instructions for Use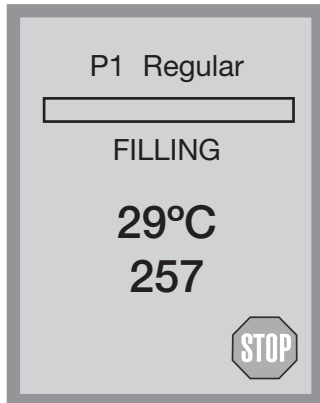

The cycle will start by draining the unit for 20 seconds. The bar on the screen indicates the progression of the cycle. The stage within the cycle is shown beneath the bar (eg. Filling, Rinsing). The cycle count and chamber temperature are displayed.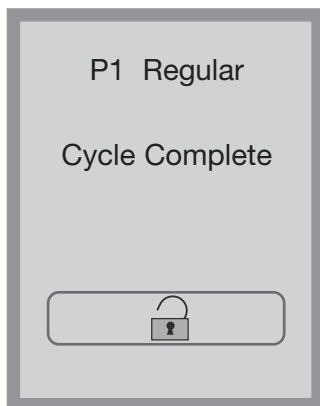

When the cycle is complete, the program name and “Cycle Complete” is displayed, and the unit beeps three times. Press  to open the door. Use gloves to unload the instruments. After all cycles, exercise caution as the instruments, trolley, baskets and cassettes may be hot. Inspect the instruments for debris prior to packaging / sterilising.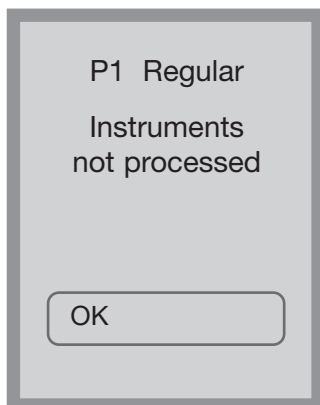

5.5 Interrupting / Aborting a Cycle


The door of the Hydrim is locked during the cycle. If the **STOP** button is pressed, the cycle is aborted and cannot be re-started. The message “Draining. Please wait” will appear. The unit will drain. Then the message “Instruments not processed” will appear. Press OK. Open the door by pressing . Use gloves if handling the instruments. Exercise caution as instruments, trolley, baskets and cassettes may be hot. **The cycle must be repeated to ensure that instruments are properly processed.** Note that if the drying phase of the cycle is interrupted, the critical parts of the cycle are complete and the message displaying the program name and “Cycle Complete. Drying interrupted” will appear.

5. Instructions for Use

5.6 Overview of Hydrim M2 Menus

The Hydrim has been set up with default values that you may wish to change (e.g. local time, language etc.). This section shows you how to adjust the Hydrim to your personal preferences.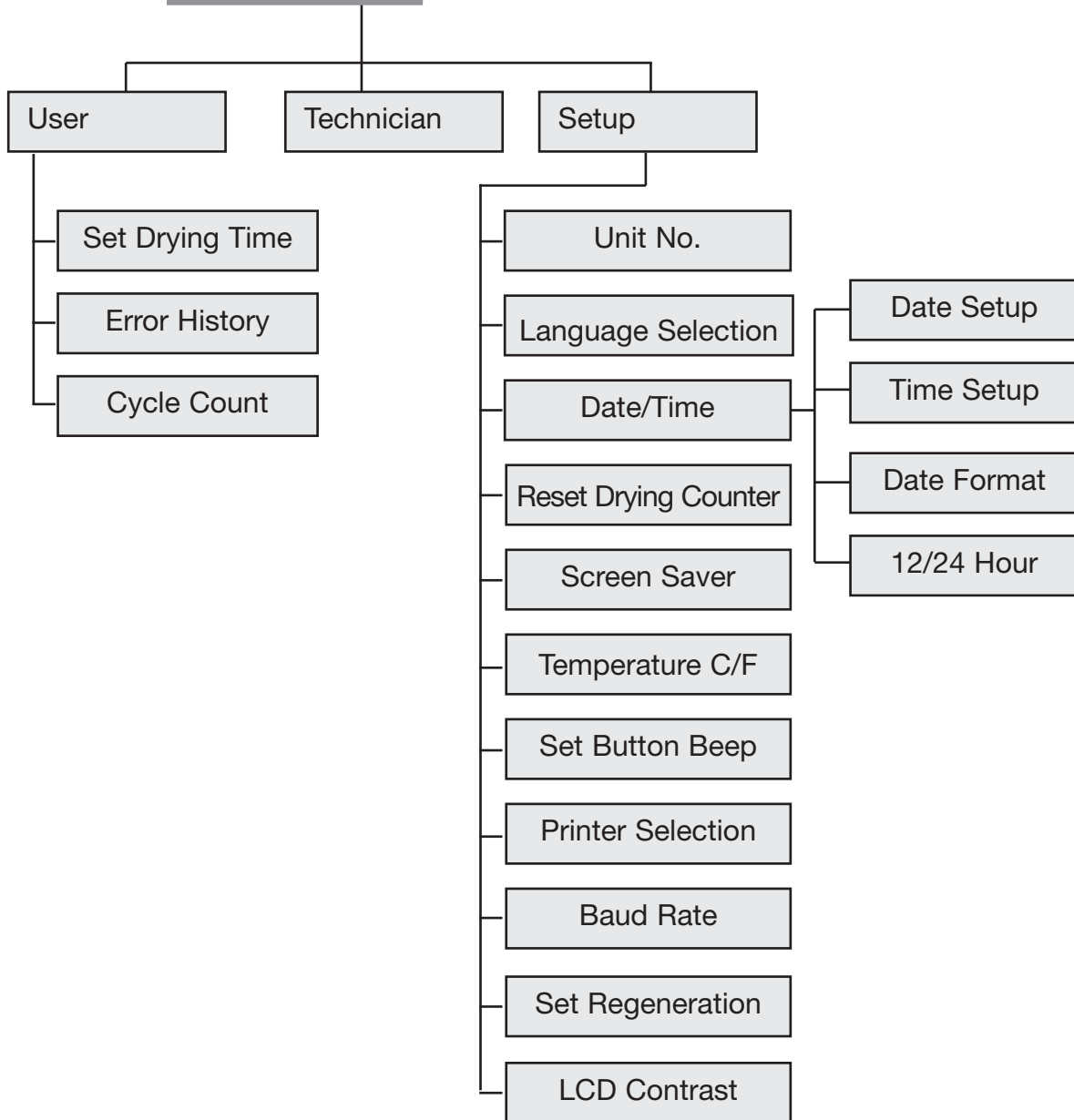

5. Instructions for Use


5.7 User Menu


To access the user menu, follow these steps:

Turn the Hydrim OFF and then ON. An *i* (information) will appear briefly at the bottom right hand of screen.


Touch the *i* to move to the Menu screen.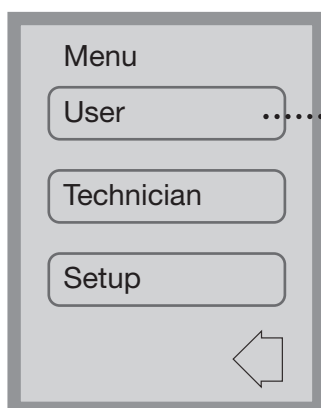

Select User.

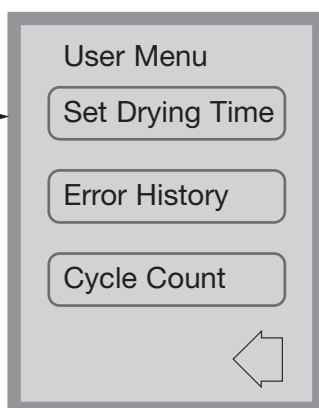
The user menu consists of three choices.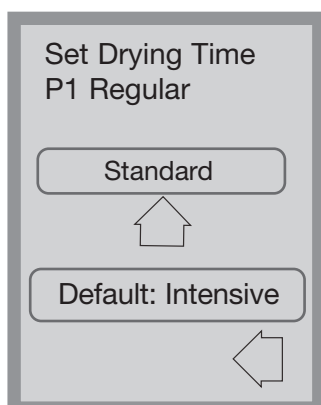

User Menu – Set Drying Time


The drying time can be set to standard, for small loads, or intensive, for larger loads. Standard drying time is 20 minutes for P1, and 15 minutes for P2 and P3. Intensive drying time is 30 minutes for P1, and 20 minutes for P2 and P3. Touch the up arrow to select standard or intensive. The default is "intensive". Touch **default** if you wish to return to the factory default. Use the back arrow to return to the previous screen.

5. Instructions for Use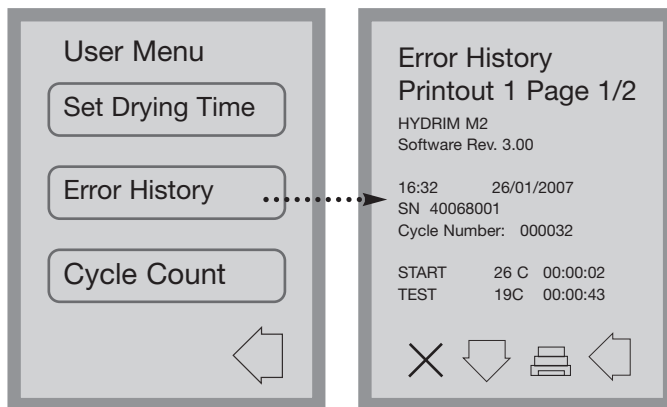

User Menu – Error History

This allows the user to see the last three errors and the last cycle completed, whether or not there was an error. Press the down arrow to move to the next page / error. If the Hydrim has a printer, press the printer icon to obtain a print-out of the error history. Press the X in the lower left hand corner to clear the error history. Press the back arrow to move to the previous menu.

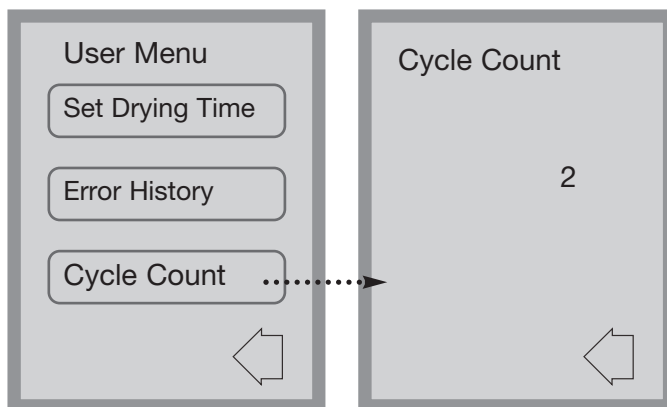User Menu – Cycle Count

This allows the user to see how many cycles have been run on the Hydrim. Touch cycle count and the number of cycles (completed and aborted) will be displayed. This cannot be reset. Touch the back arrow to return to the previous menu.

5.8 Technician Menu

This menu is only available to authorised technicians.


5. Instructions for Use

5.9 Setup Menu



To access the Setup Menu, proceed to the menu screen by turning the Hydrim OFF and then ON. An *i* (information) will appear briefly at the bottom right hand corner.


Touch the *i* to move to the Menu screen and select Setup.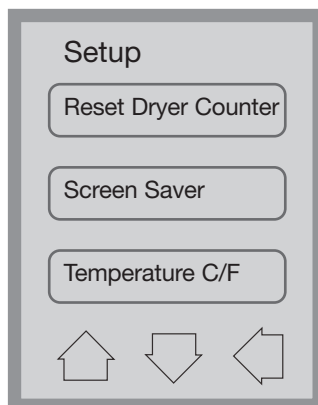

The Setup Menu consists of eleven choices as shown. Using the up and down arrows to move through the choices, touch the desired option to select it. For more detail on each setup menu option, refer to the following sections.

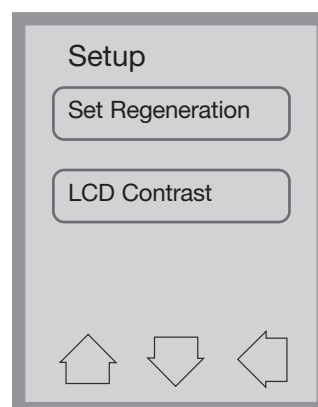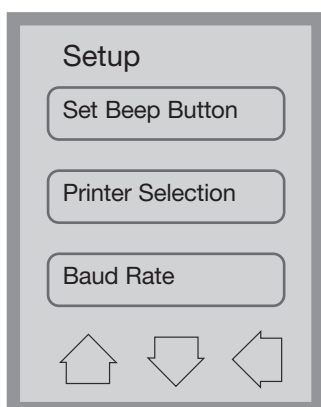
5. Instructions for Use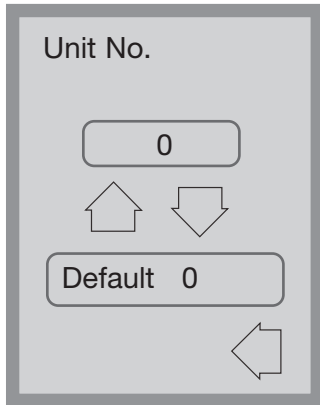



Setup Menu – Unit Number

If you have more than one Hydrim, you can designate them Unit #1, Unit #2, etc. The unit number is used for naming the printout files on the USB memory stick. Use the up and down arrow keys to select the unit number. The default is 0.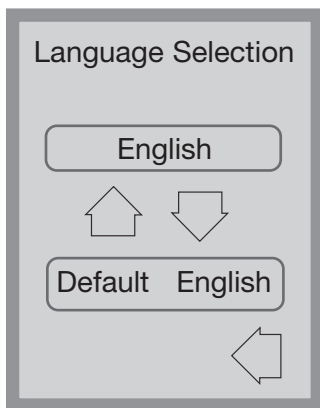

Setup Menu – Language Selection

This allows the user to select among available languages for the Hydrim. Use the up and down arrows to select the desired language. The default language is English. Touch the back arrow to return to the previous menu.

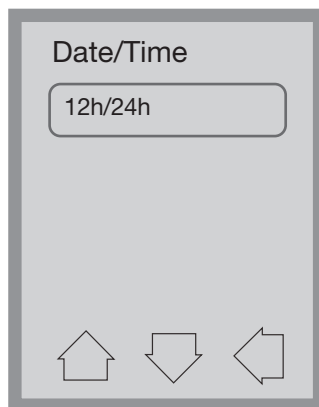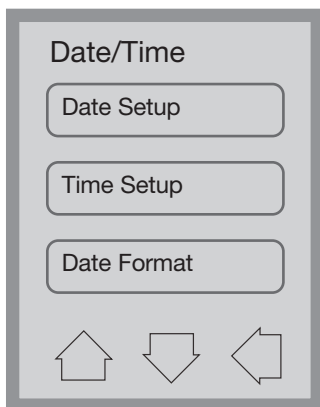Setup Menu – Date / Time

The Date/Time menu consists of four options as shown. Use the up and down arrows to move from screen to screen. The user can set the current date and time, set the date format and the time format. Touch the desired option and follow the directions below.

5. Instructions for Use


Date Setup

Enter the day, month and year and touch EN to save. Touch CL to clear the last number entered.


Time Setup

Enter the time in hours, minutes and seconds and press EN to save. Touch CL to clear the last number entered.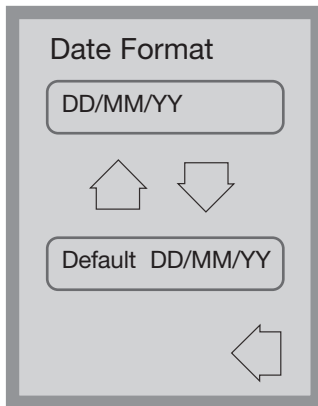


Date Format

Use the up and down arrows to change the date format from DD/MM/YY to MM/DD/YY. The default is DD/MM/YY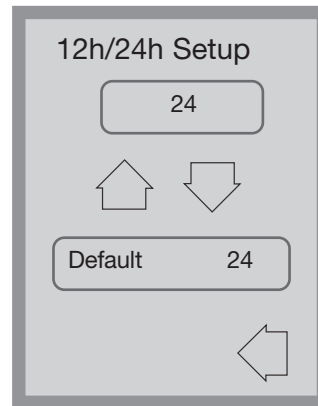

12h/24h

Use the up and down arrows to switch from 12-hour to 24-hour clock. The default is 24-hour clock.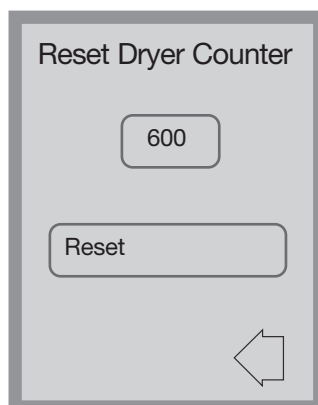


Setup Menu – Reset Drying Counter

When the “change air filter” message appears, the air filter must be changed (see section 6.4) and the drying cycle counter reset. This should be done by an authorised service technician.


To reset the drying counter, touch **Reset**. Use the back arrow to return to the previous screen.

5. Instructions for Use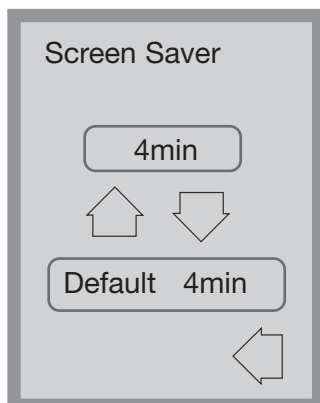


Setup Menu – Screen Saver

The user can adjust the time that elapses before the screen goes into screen saver mode. Touch **Screen Saver**. Touch the up and down arrows to select the desired Screen saver setting. The default is 4 minutes. Touch the back arrow to return to the previous menu.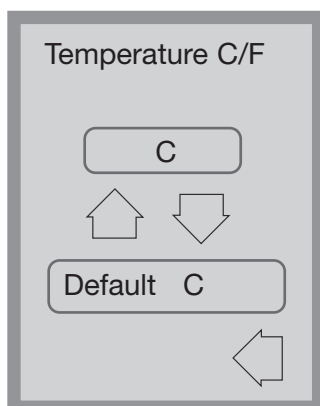

Setup Menu – Temperature C/F


The user can choose to have information displayed in degrees Celsius or degrees Fahrenheit. Touch the up and down arrows to select the desired choice. The default is degrees Celsius. Touch the back arrow to return to the previous menu.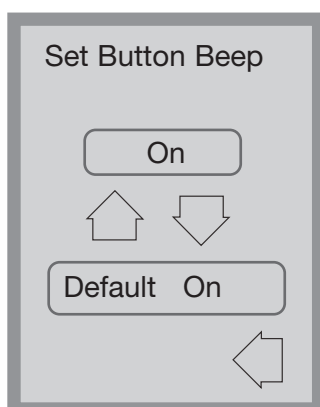

Setup Menu – Set Button Beep


The user can select between having the unit beep or not beep whenever a screen button is touched. Turn the beep on or off by using the up and down arrows. Note that the default setting is ON. Touch the back arrow to return to the previous menu.

5. Instructions for Use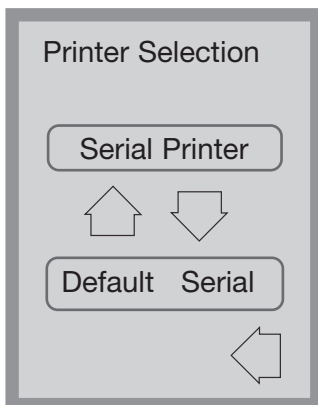


Setup Menu – Printer / Data Logger Selection

Printer selection allows the user to select either a default serial printer or the SciCan Data Logger as the recording medium for cycle information. See section 5.11 for more information.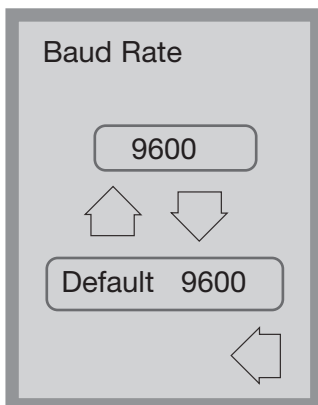


Setup Menu – Baud Rate

The Baud rate is a setting used in conjunction with the Printer Selection. Baud Rate is a measure of how fast data is moving through the serial connection. See section 5.11 for more information.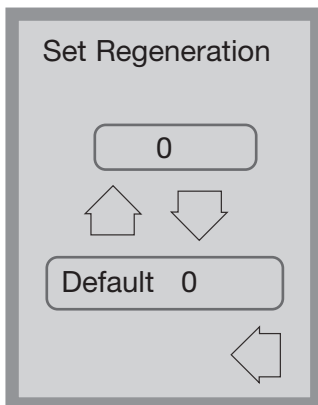

Setup Menu – Set Regeneration

Set the salt regeneration according to the local water hardness. Use the up and down arrows to change the value. The default setting is 0 . See Installation Instructions for further information.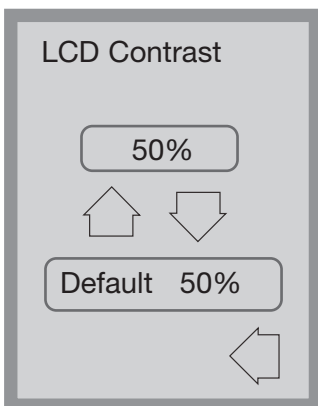

Setup Menu - LCD Contrast


The user can adjust the LCD contrast. Touch the up and down arrows to select the desired contrast. The default is 50%. Touch the back arrow to return to the previous menu.

5. Instructions for Use

5.10 Opening the Door in Case of Power Failure

If there is a power failure, ensure the machine is isolated from the main power supply. Then remove the kickplate of the Hydrim. Remove the screw with a screwdriver and lift the kickplate up and toward you. Locate the ring and pull it to open the door. Exercise caution! There may be fluid remaining in the unit and the instruments may be hot. Instruments that have not completed the cycle should not be used and should be reprocessed. When replacing the kickplate, ensure that the grounding wire is attached.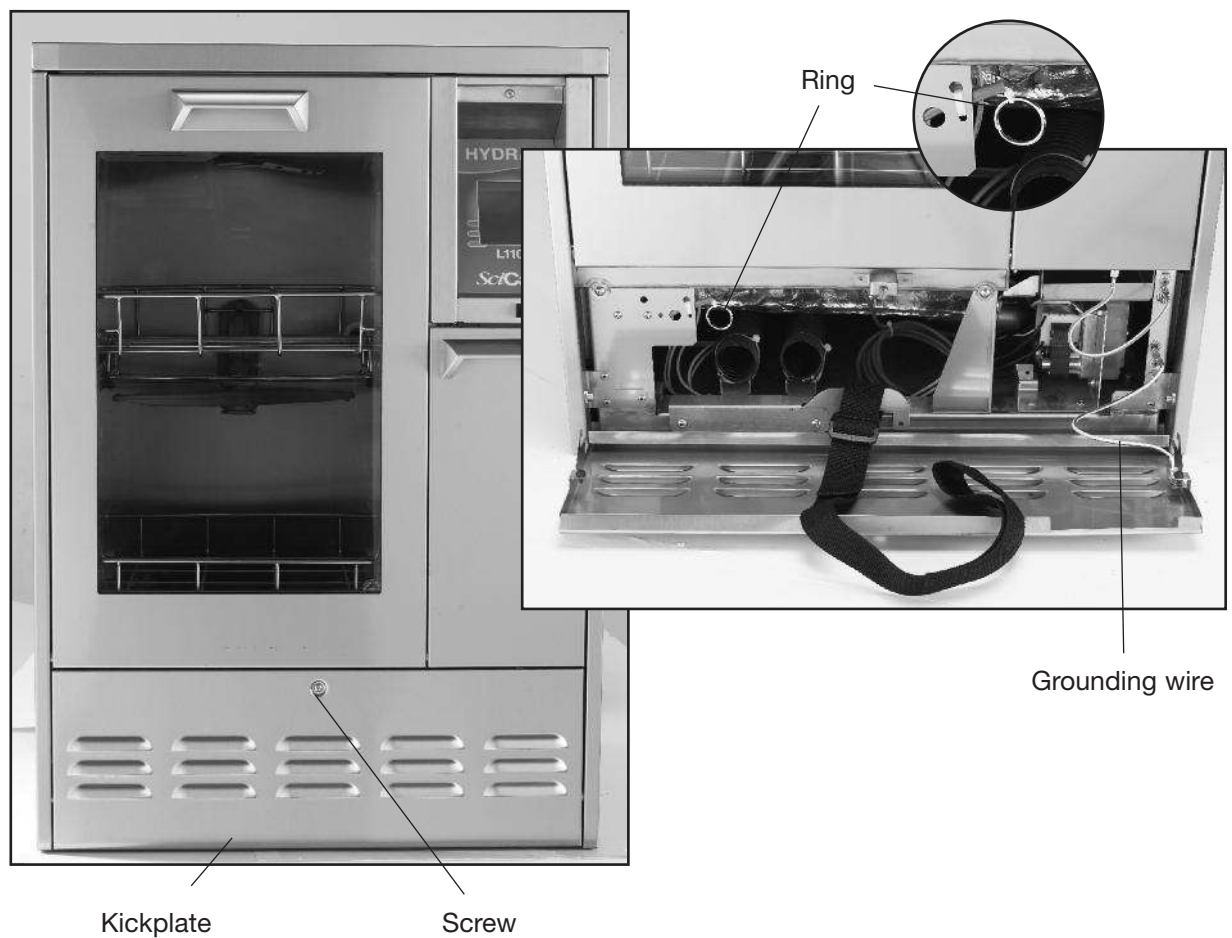

5. Instructions for Use

5.11 Printer / Data Logger Setup

The Hydrim M2 has an RS-232 port at the back, and can be used with an external printer or the SciCan Data Logger. The printers in the chart below have been tested with the Hydrim. To add or change a printer or SciCan Data Logger, follow these steps:

Turn off the Hydrim and the printer or Data Logger before connecting these devices to the unit.

1. With the printer or Data Logger connected, turn on the Hydrim and press the *i* to move to the Menu screen.
2. In the Setup Menu, select Printer Selection.
3. Select Serial Printer if connecting a printer, or USB Flash/MSD if connecting the SciCan Data Logger. Press the back arrow to return to the Setup Menu.
4. In the Setup Menu, select Baud rate.
5. Select the rate required (refer to chart below for recommended Baud rates). Use the back arrow to return to the Start screen.
6. Ensure that the date and time have been set (see Setup Menu – Date / Time)
7. Now the Hydrim will write its cycle information to the device chosen.

Printer Model	Serial Port Baud Rate
Epson TM-U220D (C31C515603)	9600
Citizen IDP-3110-40 RF 230B	9600
Star Micro SP212FD42-230	9600
Star Micro SP216FD42-230	9600
Star Micro SP512MD42-R	9600
SciCan Data Logger	Serial Port Baud Rate
For Mass Storage Device	9600

6. Care and Maintenance

6.1 Replacing the HIP Cleaning Solution

The Hydrim M2 uses a 3.8L / 1 gallon container of HIP Cleaning Solution (Part number CS-HIPL). Use only the recommended cleaning solution and read the MSDS before inserting into the unit.


To order additional HIP Cleaning Solution, contact your local dealer. When the message “no detergent” appears on the screen, the cleaning solution container is empty and is in need of changing. To avoid spillage, replace the solution container only when the message appears and be sure to wear gloves.

NOTE: Each cleaning solution container will last approximately 25-30 cycles, depending on the machine configuration.

To replace the cleaning solution container, follow these steps:

1. Open the cleaning solution door.
2. Disconnect the cleaning solution connector. Remove any dried chemical by immersing the connector in a cup of water.
3. Remove the empty cleaning solution container and discard or recycle.
4. Connect new container and insert into the cleaning solution compartment. Follow the instructions printed on each cleaning solution container.
5. Close the door.
6. Press OK. The machine display will indicate "Priming. Please wait". When the cycle selection menu appears, priming is complete and the machine is ready for use.

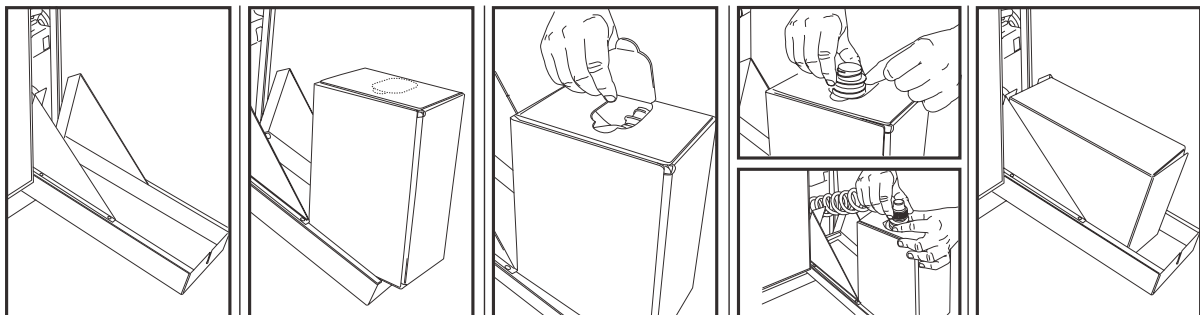6. Care and Maintenance

salt container

6.2 Refilling the Water Softening Salt Reservoir

When the error message "salt level low" appears, the salt reservoir is in need of refilling. Ensure that your authorised technician has set the water softener according to the hardness level in your area. If the level is set at 0, no message will appear and salt is not required.

To fill the salt reservoir, follow these steps:

1. Unscrew the salt container lid.
2. Fill the salt container to the top (maximum 1 kg / 2.2 lbs.)
3. Screw the salt container lid on tightly.
4. Run a complete cycle, with no instruments.

6.3 Changing the Air Filter

When the message "Replace Air Filter" appears, the HEPA filter is in need of changing. To continue using the Hydrim, press OK. Call for service as soon as possible. The air filter should be changed before 600 cycles.

6. Care and Maintenance

6.4 Cleaning the Hydrim M2

To clean outer surfaces and the decal covering the touchscreen, use a soft cloth moistened with soap and water or a surface disinfectant. Do not use harsh cleaning chemicals.

If preparing the unit for transport or storage, waste water should be removed from the sump.

Waste water in the unit may contain biological contaminants. Use a mechanical means to siphon the contents. Wear disposable rubber gloves. Dispose of absorbent material according to biological waste disposal regulations.

6.5 Filter and Wash Arm Maintenance

Filter Maintenance:

Inspect the filter in the bottom of the chamber **daily** for debris and clean if necessary. To clean, remove the filter, rinse under a tap and replace. Ensure that the filter is firmly locked into position when replaced.

Wash arms

Wash Arm Maintenance:

If the wash arms are not turning easily, remove them, rinse under a tap and reassemble. To remove the upper or middle arm, unscrew the collar counterclockwise. To remove the lower arm unscrew the collar clockwise.

Filter

7. Troubleshooting

Problem	Possible Causes and Solutions
Instruments are not clean	<ul style="list-style-type: none"> • Do not overload cassettes or instrument baskets. • Do not load too many cassettes into the Hydrim. • Check filters and wash arms for debris.
Touchscreen / Unit does not come on	<ul style="list-style-type: none"> • Ensure that the power button is turned ON. • Ensure that the plug is properly inserted into the wall socket. • Check if a fuse or circuit is blown – call service for assistance.
The wash arms do not rotate	<ul style="list-style-type: none"> • Remove the wash arms, clean and reinstall them.
The door cannot be opened	<ul style="list-style-type: none"> • The cycle is not complete. • The cycle was aborted and the unit has not completed draining yet.
Operating cycles appear to be taking too long to complete	<ul style="list-style-type: none"> • The water is not hot enough. Check that the incoming hot water temperature is in the recommended range. • Check that the hot and cold hoses are not reversed.
“No detergent” message comes on when there is still chemical left in the container	<ul style="list-style-type: none"> • Check for a kink in chemical tubing. • There is insufficient chemical to complete the next cycle. Replace the detergent box.
Drying Performance is inadequate	<ul style="list-style-type: none"> • Increase the drying time (user menu). • Change the HEPA filter. • Use "Intensive" rather than "Standard" drying • Do not overload cassettes or instrument baskets. • Check wash arms for debris.
There is a “printing” message at the end of the cycle, but there is no printer or datalogger attached	<ul style="list-style-type: none"> • In the Setup Menu, Printer / USB setup (section 5.11) ensure that "serial" is selected.
Printer does not print	<ul style="list-style-type: none"> • Check that printer is plugged into both the Hydrim and the electrical outlet. • Check that the printer is ON and has paper • Check that printer and baud rate are set correctly (section 5.11).

7. Troubleshooting

Error Messages

Error	Cause	Solution
CF1 Water Heating failure	The water does not reach the required temperature in the specified time	Call for service.
CF2 Chamber Filling Failure	The water does not fill the chamber within the specified time.	Check that the water hoses are not kinked. Open the water shut-off valves.
CF3 Chamber temperature reading failure	Faulty sensor.	Call for service.
CF4 Water evacuation failure	Water cannot drain from the unit.	Check that the drain hose is not kinked. Check that the filters in the bottom of the chamber are not obstructed.
CF5 Disinfection Failure	Temperature dropped below disinfection temperature during the disinfection phase.	Call for service.
CF6 Serial Communication Failure	Failure to read the second temperature sensor within 10 sec.	Call for service.
CF8 Secondary Chamber Temperature Sensor Failure	Second temperature sensor readings are out of limits.	Call for service.
CF9 Software or PCB Failure	Cycle length is more than 3hr.	Call for service.
CF10 Pressure Sensor Failure	Faulty sensor.	Call for service.
CF13 Temperature Validation Error	Second temperature sensor readings are out of limits.	Call for service.
CF14 Flow Switch - Error	Detergent dosing validation error	Call for service.
CF15 Chamber overflow	Overflow switch did not turn off after 30 sec of running the drain pump.	Check if drain is blocked; Call for service.
CF16 No pressure	Water pressure drops during washing or disinfection.	Call for service.

8. Material Safety Data Sheet

SECTION I - PRODUCT IDENTIFICATION

Product Name: HIP (Hydrim Cleaning Solution with Instrument Protection)
Product Use: Cleaner for Hydrim automatic instrument reprocessor machines
WHMIS Class: Class D - Division 2B
TDG Classification: Not regulated as dangerous goods.
Manufacturer/Supplier: SciCan Ltd.
1440 Don Mills Road
Toronto, ON M3B 3P9 CA
Phone: 1-800-667-7733

LEGEND HMIS		Health	/	1
Severe	4	Flammability		0
Serious	3	Physical Hazard		
Moderate	2	Personal Protection		B
Slight	1			
Minimal	0			

SECTION II - HAZARDOUS INGREDIENTS

<u>Ingredient(s)</u>	<u>CAS #</u>	<u>Wt%</u>	<u>ACGIH-TLV</u>	<u>LC50</u>	<u>Oral LD50</u>
N-(n-octyl)-2-pyrrolidone	2687-94-7	1 - 5	Not established	Not available	2050 mg/kg rat
Sodium xylene sulphonate	1300-72-7	3 - 7	Not established	Not available	7200 mg/kg rat
Phosphonic acid, (1-hydroxyethylidene)bis-, tetrapotassium salt	14860-53-8	5 - 10	Not established	Not available	520 mg/kg rat

SECTION III - PHYSICAL DATA

Boiling Point:	Not available	Specific Gravity:	1.113 (H2O = 1)
Freezing Point:	Not available	Coefficient of Water/Oil Distribution:	Not available
Vapour Pressure:	Not available	% Volatile:	Not available
Vapour Density:	Not available	Evaporation Rate:	Not available
Solubility in Water:	Complete	pH:	9.3 - 9.8
Physical State:	Liquid	Viscosity:	Water thin
Appearance:	Clear colourless Liquid	Odour Threshold:	Not available
Odour:	Odourless		

SECTION IV - FIRE AND EXPLOSION DATA

Flammability:	Not flammable by WHMIS criteria.		
Flash Point:	> 93.3 °C (> 199.94 °F)	LEL:	Not applicable
Hazardous Combustion Products:	May include and are not limited to: Oxides of carbon. Oxides of sulphur. Oxides of phosphorus.		
Autoignition Temperature:	Not applicable		
Explosion Data:			
Sensitivity to Mechanical Impact:	Not applicable.		
Sensitivity to Static Discharge:	Not applicable.		
Means of Extinction:	Treat for surrounding material.		
Special Fire Hazards:	Firefighters should wear a self-contained breathing apparatus.		

SECTION V - REACTIVITY DATA

Conditions for Chemical Instability:	Stable under recommended storage conditions.
Incompatible Materials:	Do not mix with any other cleaning or disinfecting product.
Reactivity:	Do not mix with any other cleaning or disinfecting product.
Hazardous Decomposition Products:	None known.

SECTION VI - TOXICOLOGICAL PROPERTIES

Route of Entry: Eye, Skin contact, Ingestion.

EFFECTS OF ACUTE EXPOSURE:

Eye: May cause irritation.

8. Material Safety Data Sheet

Skin: May cause irritation.
Inhalation: Not a normal route of exposure.
Ingestion: Like any product not designed to be ingested, this product may cause stomach distress if ingested in large quantities.

EFFECTS OF CHRONIC EXPOSURE:

Skin: Prolonged or repeated exposure can cause drying, defatting and dermatitis.
Irritancy: May cause irritation.
Respiratory Tract Sensitization: Non-hazardous by WHMIS criteria.
Carcinogenicity: Non-hazardous by WHMIS criteria.
Teratogenicity: Non-hazardous by WHMIS criteria.
Mutagenicity: Non-hazardous by WHMIS criteria.
Reproductive Effects: Non-hazardous by WHMIS criteria.
Synergistic Materials: Not available

SECTION VII - PREVENTATIVE MEASURES

Gloves: Natural or butyl rubber, nitrile or neoprene gloves.
Eye Protection: Wear safety glasses with side shields.
Respiratory Protection: Not normally required under normal use conditions.
Other Protective Equipment: As required by employer code.
Engineering Controls: General ventilation normally adequate.
Leak and Spill Procedure: Small spills (<5 gallons) can be flushed into sewage system in accordance with local regulations. Prevent large spills (>5 gallons) from entering sewers or waterways. Use non-reactive absorbent and place in suitable, covered, labelled containers. Contact emergency services and supplier for advice.
Waste Disposal: Review federal, provincial, and local government requirements prior to disposal.
Handling Requirements: Use good industrial hygiene practices in handling this material. When using do not eat or drink. Wash hands before breaks and immediately after handling the product.
Storage Requirements: Keep out of reach of children. Store in a closed container away from incompatible materials.

SECTION VIII - FIRST AID

Eye: Flush with cool water. Remove contact lenses, if applicable, and continue flushing. Obtain medical attention if irritation persists.
Skin: Flush with cool water. Wash with soap and water. Obtain medical attention if irritation develops or persists.
Inhalation: No specific first aid measures are required.
Ingestion: Do not induce vomiting. Never give anything by mouth if victim is unconscious, or is convulsing. Obtain medical attention.

SECTION IX - PREPARATION INFORMATION

Effective Date: 15-May-2009

Expiry Date: 15-May-2012

Prepared By: Dell Tech Laboratories Ltd. (519) 858-5021

Information contained herein was obtained from sources considered technically accurate and reliable. While every effort has been made to ensure full disclosure of product hazards, in some cases data is not available and is so stated. Since conditions of actual product use are beyond control of the supplier, it is assumed that users of this material have been fully trained according to the requirements of all applicable legislation and regulatory instruments. No warranty, expressed or implied, is made and supplier will not be liable for any losses, injuries or consequential damages which may result from the use of or reliance on any information contained in this document.

9. Warranty

Limited Warranty

For a period of one year, **SciCan** guarantees that the **Hydrim M2**, when manufactured in new and unused condition, will not fail during normal service due to defects in material and workmanship that are not due to apparent abuse, misuse, or accident. In the event of failure due to such defects during this period of time, the exclusive remedies shall be repair or replacement, at **SciCan's** option and without charge, of any defective part(s), provided **SciCan** is notified in writing within thirty(30) days of the date of such a failure and further provided that the defective part(s) are returned to **SciCan** prepaid.

This warranty shall be considered to be validated, if the product is accompanied by the original purchase invoice from the authorized **SciCan** dealer, and such invoice identifies the item by serial number and clearly states the date of purchase. No other validation is acceptable. After one year, all **SciCan's** warranties and other duties with respect to the quality of the product shall be conclusively presumed to have been satisfied, all liability therefore shall terminate, and no action or breach for any such warranty or duty may thereafter be commenced against **SciCan**.

Any express warranty not provided hereon and any implied warranty or representation as to performance, and any remedy for breach of contract which, but for this provision, might arise by implication, operation of law, custom of trade or course of dealing, including any implied warranty of merchantability or of fitness for particular purpose with respect to all and any products manufactured by **SciCan** is excluded and disclaimed by **SciCan**.

10. Specifications

Height, freestanding: 850mm

Height, built-in: 830mm

Width: 600 mm

Depth: 600 mm

Depth with door open: 1200 mm

Weight: 80 kg

Running Noise: 60 dB(A)

Hot and cold water connections: G 3/4"

Inlet water pressure: 2-5 bar

Incoming hot water temperature: 60° - 84°C

Drain: 3/4"

Water softener: 1.0 kg salt capacity

Electrical Connection: 230-240 V 50 Hz 13 A

Equipment pollution degree: Pollution Degree 2

Equipment installation category: Installation category II

Maximum relative humidity: 80% for temp up to 31°C
50% for temp up to 40°C

Operating temperature range: +5°C to 40°C

Max. altitude: 2000m

Mains supply: + / -10% of nominal

Required clearance: 10mm on either side, at the top and at the rear of the unit.

11. Serial Number Plate

When ordering supplies, spare parts or requesting service, please ensure that the information contained on the serial number plate is available (Model number, serial number etc.).

The serial number plate is located at the bottom left on the rear panel of the Hydrim M2 unit. A small label is located on the left hand side of the chemical door.

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Fabriqué par:

SciCan Ltd

1440 Don Mills Road
Toronto ON M3B 3P9
Canada

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